

Introducing TheyFit™, the world's first sized-to-fit condom line in 70 custom-fit sizes for maximum pleasure & safety.

CONDOMania® FIT KIT™



To find your ideal condom, follow the instructions below to measure your Johnson and identify your custom size.

INSTRUCTIONS:

1 Cut on dotted lines to remove and separate alphabetical length measurement from numeric width measurement.

2 Crease length measurement strip down center to create a V-like shape. Place erect penis down center only as far as condom would unroll to find your letter.

3 Next, wrap numeric fit strip around middle of erection to find your corresponding number.

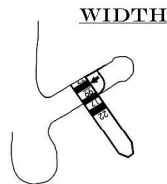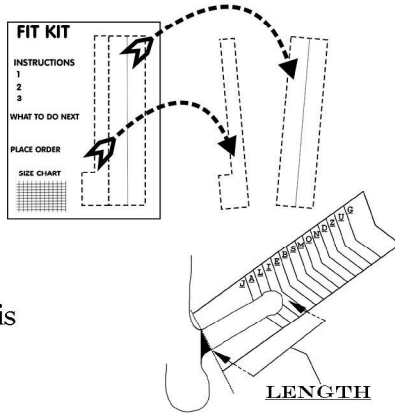

WHAT TO DO NEXT:

Refer to the Fit Kit Chart below to find your individual condom size! Your custom fit is shown where your alphabetical length row meets your numeric width column. If you find that your size has an arrow in the box, follow the arrow to the next best size. If your size box is empty, please call us for additional assistance.

Please note: your "custom size" as determined by this Kit is a suggested size. You may want to choose a slightly different size based on your individual preference.


NOW PLACE YOUR ORDER!

Visit us at Condomania.com/TheyFit or call 1-800-926-6366 (323-969-0102)

SIZE CHART

	WIDE							WIDER					
	33	44	55	66	77	88	99	11	17	21	22		
L													
O	J33	J44	J55										
N	A33	A44	A55	A66									
G	L33	L44	L55	L66	L77	L88	L99						
I	I33	I44	I55	I66	I77	I88	I99						
E	E33	E44	E55	E66	E77	E88	E99	E11	E17				
B		B44	B55	B66	B77	B88	B99	B11	B17				
S				S66	S77	S88	S99	S11	S17				
M					M77	M88	M99	M11	M17				
L					O77	O88	O99	O11	O17				
O						N88	N99	N11	N17	N21			
N									D11	D17	D21		
D										Z11	Z17	Z21	
G											U17	U21	U22
R												G21	G22

@ HERE
MEASURE WIDTH at ARROW

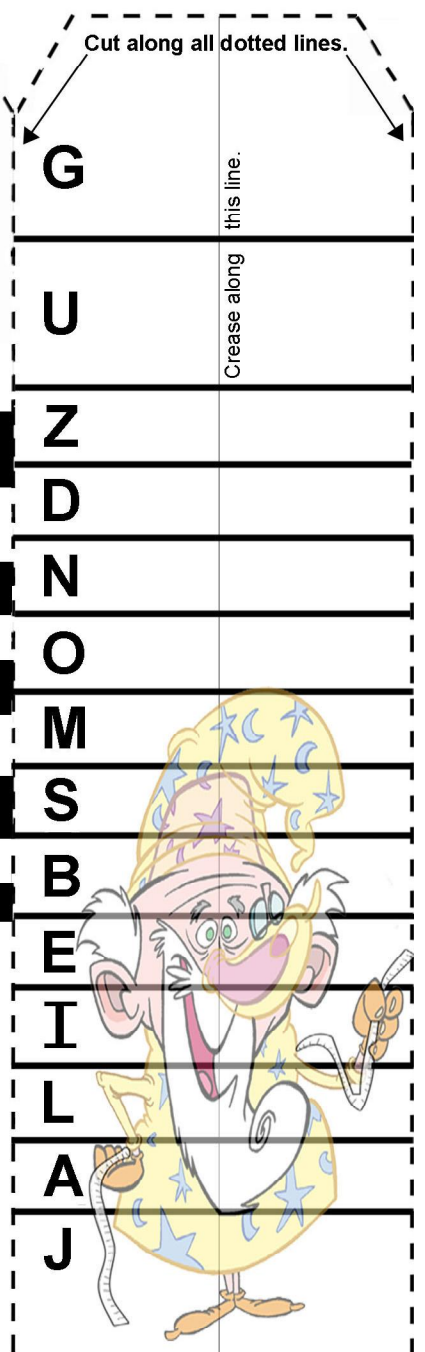

MEASURE LENGTH

Please be careful when using this Fit Kit. Watch out for paper cuts! User assumes all risk of bodily injury associated with usage.

PLACE UNDER ERECTION ONLY DOWN AS FAR AS CONDOM WOULD UNROLL
Measurement begins at this line, cut before using.

TheyFit™ Condoms & The Fit Kit™ are produced and marketed under patents by Oceans Seven Int'l.

The media is talking about TheyFit condoms:

"Once you have been custom-suited, an off-the-rack condom feels like punishment."

--*Men's Health Magazine*

"Our tester went wild for TheyFit, 'It's like wearing nothing at all. These are, seriously, the best I've ever used.'"

--*Cargo Magazine*

"Custom condoms are worth the extra effort... so from here on out, it's condom couture for me."

--*New York Post*

TheyFit Condoms: Perfect score of 100 and the highest rating of "Excellent."

--*Consumer Reports* (02/2005)

"Given the vast differences in penis dimensions, the one-size-fits-all approach is unrealistic."

--*Dr. Michael Reece, Indiana University*

Customers are talking about TheyFit condoms:

"You guys are saving lives! Seriously!! A lot of guys who can't use regular condoms just don't use them! Me included. Now, I have an alternative. Thank you."

"It was the best fit I have ever found. I've tried many sizes and Brands and never had a good experience. These fit perfectly and provided the sensitivity I have always looked for."

"Best fitting condom I've ever had. It was easy in fact, to forget it was on."

"Didn't have to worry about slipping. Condom quality was evident when I took one out when shipment arrived."

"This product line is far superior to any condom I have ever used in over 26 years."

"Changed my sex life. I was always suffocating in other brands or they were wide enough but too long. They Fit was just what I was hoping for."

"Excellent product that allows the full use of my penis in the love making experience. I have used condoms for 27 years as a method of birth control. Thanks for this TheyFit brand."

"First time i used it, i didn't tell my girlfriend about it. She did feel the difference. We both like the comfort of TheyFit."

"For the first time, after making love, I didn't have a bruised purple ring around the base of my penis from the condom's ring. Also, my partner, for the first time, was able to comfortably put the condom on me."

"I kept checking to see if it was still on, because for the first time, a condom wasn't soooo tight that it cut off my circulation. My wife couldn't tell it was on and thought I forgot to use one."

"Before TheyFit, I was always reluctant... due to [my] small size and the fact that condoms would just come off. TheyFit have given me a significant boost in my confidence level and I no longer fear situations in which I [need to use a condom]."

For more information about TheyFit condoms, please visit Condomania.com or call 1-800-926-6366